



# COVID-19 FACE SHIELD



*The COVID-19 Face Shield is manufactured from an open-source design. Design details & other useful information are found at [www.prusa3d.com/covid19](http://www.prusa3d.com/covid19).*

## Face Shield is composed of 3 parts:

- Headpiece made of polypropylene
  - Adjustable Elastic Tourniquet made of non-latex material
  - Shield is polyester or vinyl\*
- \*Dependent on material supply.

## Assembly:

1. Stretch elastic band holes over side pins on head piece. Adjust to fit. Fold extra elastic over pins toward back.
2. Attach either end of clear shield to 3 pins on headpiece.
3. Vinyl shield - Attach pins in curved bottom piece to clear shield.
  - a. Polyester shield.-No bottom piece is used. (as shown above)

## Disinfecting:

This product must be disinfected using the guidelines put forth by the design company. Guidelines are available online [https://help.prusa3d.com/en/article/prusa-face-shield-disinfection\\_125457](https://help.prusa3d.com/en/article/prusa-face-shield-disinfection_125457)

**SHIPS UPS GROUND  
FREE WITHIN THE  
UNITED STATES**

Case Quantities of 25  
and 50.

Components are  
shipped together in a  
zip-locked Assembly  
instructions included.

For orders of 1,000 or more,  
please contact us directly.

[www.allmetalstamping.com](http://www.allmetalstamping.com)  
[ams@allmetalstamping.com](mailto:ams@allmetalstamping.com)  
715.223.6324

**Order Online**

[https://allmetalstamping.com  
/order-covid-19-faceshield](https://allmetalstamping.com/order-covid-19-faceshield)